# Hyabak

**EN** Moisturising and lubricating solution for eyes and contact lenses.  
**ABAK® Bottle - Preservative-free.**

## INGREDIENTS:

Sodium hyaluronate ..... 0.15 g  
Sodium chloride, actinoquinol, trometamol, hydrochloric acid,  
water for injections ..... to 100 mL

## NAME AND ADDRESS OF MANUFACTURER:

Laboratoires Théa - 12 rue Louis Blériot  
63017 CLERMONT-FERRAND - Cedex 2 - FRANCE

## WHEN TO USE HYABAK:

HYABAK is a hypotonic solution intended to be administered on the eye or on contact lenses. It contains hyaluronic acid (lubricating and moisturising agent) and actinoquinol (agent with anti-UV properties).

HYABAK is recommended:

- To moisten and lubricate the eye in case of dryness sensations or tired eyes caused by external factors such as wind, smoke, pollution, dust, dry heat, air conditioning, air travel, prolonged work in front of a computer screen.
  - For contact lens wearers, it lubricates and hydrates the lens to make insertion and removal easier and provides immediate comfort of wear during the day.
- HYABAK, thanks to the ABAK® system, can dispense drops of preservative-free solution. Therefore it can be used with any type of contact lens. The absence of preservative avoids the onset of irritation and allergic reactions. It also allows the protection of ocular tissue.

## SPECIAL WARNINGS AND SPECIAL PRECAUTIONS FOR USE:

- Do not use if you are allergic to any ingredient.
  - Avoid contact between the eye and the tip of the bottle.
  - Do not inject, do not swallow.
  - **The UV filter does not replace the wearing of sunglasses.**
- If the safety ring is damaged, do not use HYABAK.

KEEP OUT OF THE SIGHT AND REACH OF CHILDREN.

## INTERACTIONS:

Leave at least 10 minutes between using two different eye products.

## HOW TO USE HYABAK:

### DOSE:

1 drop in each eye during the day as often as needed. In contact lens-wearers: a drop on each lens when putting on and removing them, and also as often as necessary during the day.

## METHOD AND ROUTE OF ADMINISTRATION:

USE ONLY IN THE EYE.

**STERILE** **A** Follow these steps to make sure you use Hyabak properly:

- Wash hands thoroughly before use.
- Do not touch the tip of the bottle on the eye or eyelids.
- Drop one drop of HYABAK in the conjunctival sac (the space between the eye and eyelid), while gently lightly pulling the lower lid down and looking up. The Hyabak bottle takes longer to release a drop than other bottles.
- When putting on contact lenses: drop a drop of HYABAK in the lens concavity (the curved inner surface of the lens that sits on the eye).
- Close the bottle after every use.

## FREQUENCY AND TIME WHEN THE PRODUCT SHOULD BE ADMINISTERED:

Instill drops regularly during the day, as needed.

## SIDE EFFECTS:

You may have mild eye irritation, although this is very unusual.

## HOW TO STORE HYABAK:

DO NOT USE AFTER THE EXPIRY DATE CLEARLY MARKED ON THE OUTER CARTON.

Store below 25°C.

Do not keep an open bottle for more than **3 months**.

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